English

Introduction

The new coronavirus (2019-nCoV) is a coronavirus of ARN de la familia Betacoronaviridae... This test is for in vitro professional diagnostic use...

Performance Characteristics

Table with columns: Region, Test Result, Sensitivity, Specificity. Rows include Europe, South America, Korea, and USA.

Kit Components

- Each test device sealed in a foam pouch with a desiccant
- 1000 absorbent tissue
- Instructions for use

Specimen Collection and Storage

- 1. Specimen should be handled carefully as an infectious agent and should be collected by trained personnel.
2. The specimen should be collected in a sterile, leak-proof, leak-resistant, handle with care.

Assay Procedure

- 1. Preparation: Wash hands thoroughly with soap and water.
2. Do not touch the side of the foil pouch until ready to perform the test.
3. Test: Remove the aluminum seal from the assay device, immerse nasopharyngeal swab in the assay diluent...

Interpretation of Results

- [Negative] The appearance of only one red band at the control line (C) within the result window indicates a negative result.
[Positive] Two bands appearing: one at the control line (C) and one black test line (T).

Storage

- 1. Store at 2-8°C.
2. Do not use the assay device after the expiration date.

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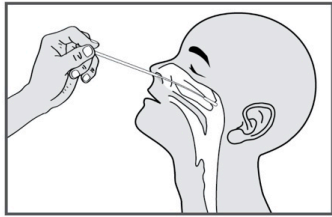
EN Specimen Collection

Nasopharyngeal Swab

- 1 Tilt the patient's head slightly backwards.
- 2 Insert a nasopharyngeal swab horizontally into the nasal cavity until resistance is met at the level of the turbinate.
- 3 Rotate the swab gently against the nasopharyngeal mucosa for 10 - 15 seconds.
- 4 Remove the swab while making sure that the tip of the swab is wet.

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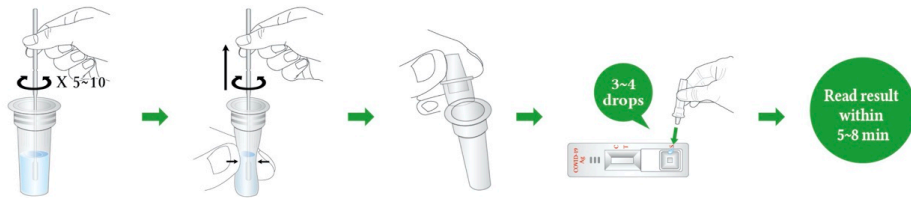
EN Assay Procedure

- 1 Insert the swab specimen and swirl the swab 5-10 times.
- 2 Remove the swab while gently squeezing the head of the swab.
- 3 Take a filter cap from the pack and close the diluent tube securely.
- 4 Invert the assay diluent tube and gently squeeze it to draw 3-4 drops (90-150µl) into a sample well on the device.
- 5 Read the result within 5-8 minutes.

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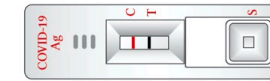
EN Interpretation of Results

Negative

One red line "C" within the result window.

Positive

Two bands ; black "T" test line and red "C" control line within the result window.

Invalid

No "C" line within the result window. It is recommended that the specimen be retested.

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